



**Turn on the
test box**



**Turn on the
Corpuls**



**Connect the
test box**



**“Mark mission
as test mission”
appears**

Mark mission as test mission?

Yes

No

**Press the
“Yes” softkey**



Pre-shift Test Procedure Cont.

“Test” will appear top left




Connect all ECG leads to test box



Press the “Manual” button



Reduce the energy to 50J

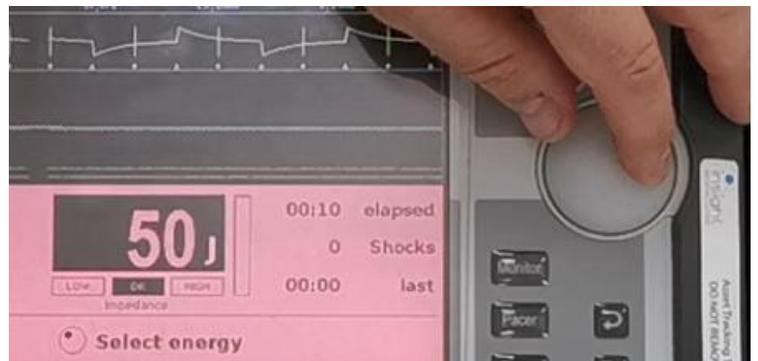

Charge at 50J and deliver the shock



Pre-shift Test Procedure Cont.

**Disconnect
all ECG leads**


**Ensure the
energy
remains at 50J**



**Charge at 50J
and deliver
the shock**




**Allow second
printout to
complete**

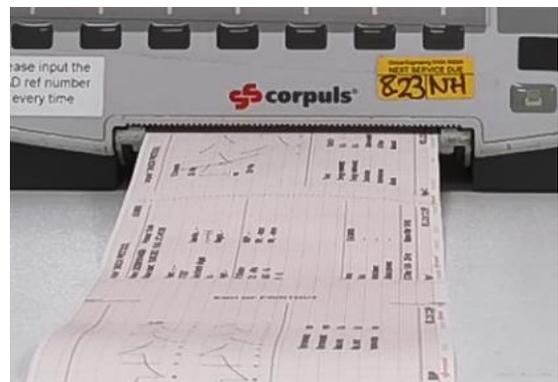

Test is complete. Turn off the Corpuls and test box, reattach the CorPatch electrodes and review the test results.


Pre-shift Test Procedure Results

Corpuls C3

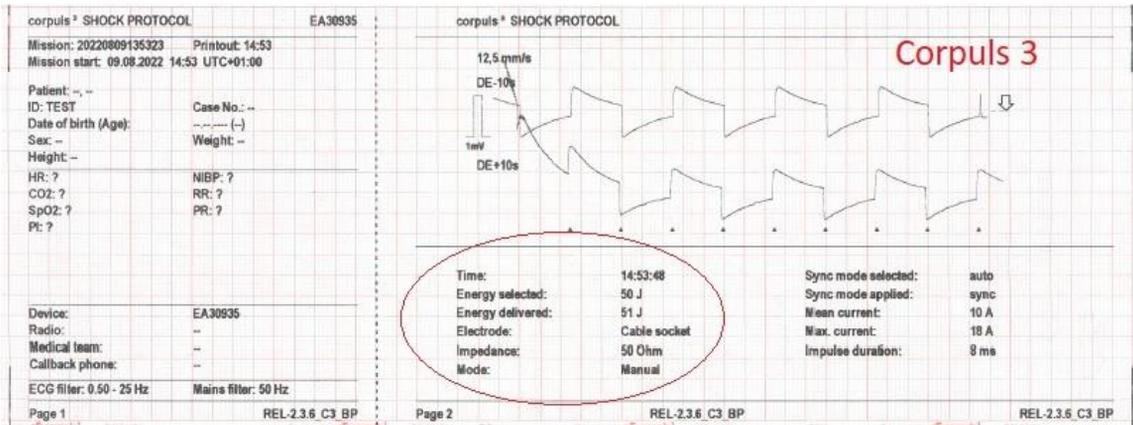

The “delivered energy” MUST be within 45J to 55J to be a successful test. 44J and below or 56J and above are unsuccessful tests.

Corpuls C3T

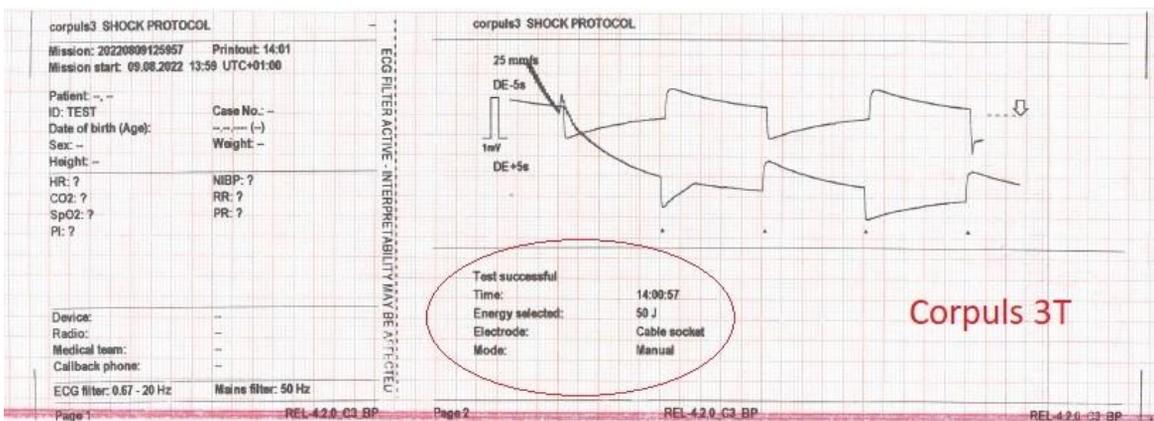

The Corpuls C3T will indicate whether the test was successful or whether it failed on the printout.

If the test was unsuccessful, please tag as faulty with reason.